Prior to coming to coming to COMP for your COVID Test:	COVID SAMPLE Check-Off list: (Did you complete the following steps prior to calling for your sample to be picked up.)
- Remember you cannot eat, drink, or rinse your mouth 30minutes prior to your test. (including chewing gum). Things to keep in mind when giving your sample: - Bubbles do not count. - Your liquid saliva must reach the saliva line in order to be viable. Tips to help you produce saliva: - Think of eating a lemons  - Think of eating sour patch kids  - Massage the back of your cheeks clockwise to produce more saliva. 	○ Fill the collection tube with your saliva to the saliva line, not counting bubbles. ○ Unscrew the funnel once done giving sample, and put used funnel inside large biohazard bag. ○ Tightly screw the blue cap on your sample. ○ Shake your sample collection tube to mix your saliva and the solution for 5 seconds. ○ Place the sealed sample tube into the small sample bag, letting out any air and leaving in the absorption pad. ○ Place the small bag with your sample inside the plastic kit provided. ○ Place the kit inside the large biohazard bag where your funnel went and seal tightly. ○ Drop your sample inside the biohazard container designated for samples. ○ Call 951-222-2208 extension 1179, to notify that your sample is ready to be picked-up.

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